



16 April 2015

## **User release notes for HearID 5.1.9.3**

### **Who is HearID 5.1.9.3 for?**

HearID 5.1.9.3 provides minor updates to the TEOAE measurement modules and to the HearID patient management system.

HearID 5.1.9.3 is intended:

- For new HearID systems running with the Mimosa Acoustics USB-connected Audio Processing Unit.
- As an upgrade to HearID systems currently running v5.1 versions.
- As an upgrade to HearID systems running earlier versions, excluding the TEOAE module.
- Older HearID systems with the TEOAE module should only be upgraded after consultation with Mimosa Acoustics. It is usually possible to run old and new systems side-by-side on the same computer, so we recommend installing a new version rather than upgrading an older version.

### **TEOAE v3.5.7 release notes**

- TEOAE measurements can now be paused and continued. Use this feature when testing in noisy environments or if you need to adjust or communicate with the patient. However, if the probe is adjusted or refitted, the measurement must be stopped, recalibrated, and restarted otherwise the stimulus level may have changed.

### **HearID 5.1.9.3 release notes**

- Measurement files associated with the MOCR measurement are now automatically included in system backups.